Instructions for use

AGFS Forano VAC AGSS Forano VAC SEGA Forano VAC

	CE
Deutsch.....	2
English.....	9
Français.....	16
Italiano.....	23
Nederlands.....	30
Polski.....	37
Español.....	44
Magyar.....	51

AGFS Forano VAC

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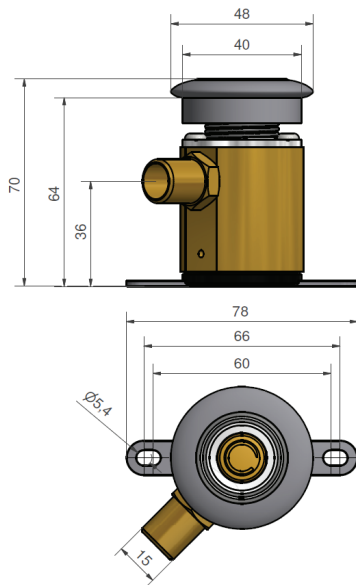
Dear customers,

Thank you for purchasing this Greggersen product.

If you have any questions or require information, please
contact our sales and support team.

Tel.: +49-(0)40 739357-0

E-mail: sales@greggersen.de

Please read the instructions for use thoroughly before using
the medical device. Only use it once you have completely
understood the application and function!

Medical device in
accordance with:

DIN EN ISO 7396-2

DIN EN ISO 9170-2

VMP 2017-745

Preface

The AGSS Forano terminal unit from Greggersen Gasetechnik GmbH complies with the relevant provisions of Regulation (EU) 2017/745 of the European Parliament and Council on medical devices (MDR 2017/745) and is classified as a Class I non-active, non-invasive device.

Intended use

To transfer residual gases during inhalation anaesthesia at a suitable disposal point.

Functionality

The vacuum-operated AGSS (anaesthetic gas scavenging system) is connected to a suction device according to ISO 7396-2. If the terminal unit is activated by coupling the plug, the excess anesthetic gases are sucked off by the resulting negative pressure and passed into a disposal device.

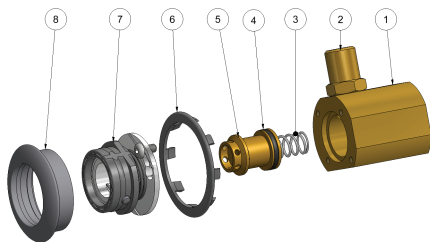
The extraction through the terminal unit is started by coupling an AGSS connector or a hose with the appropriate connection. If the connection is decoupled again, the suction is interrupted.

Warnings

- Prior to the use of the device, the user must be familiar with the product and the instructions for use and has to have understood them.
- The terminal unit must be checked for external damage or contamination before each use.
- When cleaning, make sure that no liquids enter the terminal unit. After cleaning, the terminal unit must be dried completely, before it is put back into operation.
- The terminal unit may only be connected to a suction system that has been designed in accordance with ISO 7396-2!
- After installation of the terminal unit, an acceptance test according to ISO 7396-2 must be performed.
- Only use the terminal unit with type 1L AGSS connections.
- All serious incidents must be reported to Greggersen and to the health authority responsible for the place of residence.

Device description

Pos.	Description
1	Casing AGSS
2	Fitting 15 mm
3	Closing spring
4	O-Ring 15x2,5
5	Closing piston
6	Panel ring
7	Front part Forano
8	actuator

Accessories

- 902071 plug AGSS
- 902095 AGSS-plug, angled 45°

Patient target group

The patients themselves are not in direct contact with the product, the patient target group is therefore defined by the device connected.

When using the AGSS Forano VAC as intended, the focus is not on protecting the patient but rather on protecting persons in the immediate vicinity. The medical personnel in particular are affected.

Intended users

The medical device is intended for clinical use by medically qualified personnel (therapists, care personnel, doctors).

Operation

Before starting operation each time, an external visual inspection and a functional test have to be carried out. After visual inspection for obvious damage or contamination, the actuator must be activated and smooth movement verified.

To establish a connection, an AGSS plug or a hose with an AGSS type 1L connection is inserted into the terminal unit. When the connector is engaged, you will feel a distinct snap.

By pressing the actuator on the terminal unit, the connector is released from the terminal unit. The plug should be held by hand! The suction is interrupted.

Cleaning

The terminal unit can be wet cleaned with commercially available disinfectants and cleaning agents. When cleaning, make sure that no liquid enters the terminal unit. The terminal unit must be completely dry prior to subsequent use. The hygiene plan of the healthcare facility must be observed.

Installation

- Solder the base block to the piping system.
- Attach the front panel.
- Place the front part of the gas outlet and fix it with screws.
- Click on the actuator.
- Establish the vacuum connection.

Caution!

During soldering, the lines must be purged with inert gas.

Caution!

The terminal unit Forano AGSS VAC may only be connected to suction systems that have been designed in accordance with ISO 7396-2.

Caution!

After assembly, an acceptance test in accordance with ISO 7396-2 must be carried out. The installation may only be performed by trained personnel.

Maintenance and inspection

Always perform a visual inspection for external damage before using the terminal units (external damage). A function and tightness check should be performed at least once a year (if necessary replacement of wearing parts), maintenance should be carried out every six years (measurement and safety check) in particularly high-use areas more frequently!

Faults and damage must only be dealt with by authorised expert personnel. Only use genuine GREGGERSEN replacement parts when performing repairs.

Disposal

The terminal unit is to be disposed of in accordance with the relevant national regulations.

Technical Data

Design:	All-metal design, single-handed operation for connecting/disconnecting.
Connection:	<u>Supply lines</u>
	Exhaust line: Cu pipe 15x1
Throughput:	min. 50 l/min at 500 kPa compressed air
Installation:	Ceiling supply systems
	Installation dimensions - see attachment

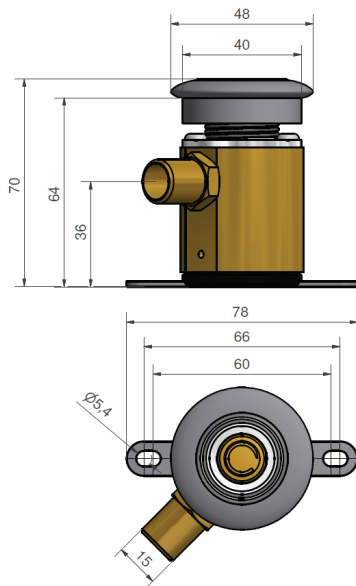
Our products are partly made of brass with a lead content of more than 0.1%. As an alloy component, lead is firmly bound in the material so that no exposure is to be expected.

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GREGGERSEN Gasetechnik GmbH / Bodestr. 27-29 / 21031 Hamburg / Germany