GEBRAUCHSANWEISUNG INSTRUCTIONS FOR USE MODE D'EMPLOI AIRMOTOR FORANO

Deutsch	2
English	5
Français	8

Entnahmestelle Airmotor Forano

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Outlet Airmotor Forano

Dear customers,

Thank you for purchasing this Greggersen product. If you have any questions or require information, please contact our sales and support team.

+49-(0)40 739357-0, sales@greggersen.de

Please read the instructions for use thoroughly before using the device. Only use it once you have completely understood the application and function!

Technical data

Design:	All-metal constructions, stainless steel housing with front panel, one-handed coupling and uncoupling, release via female connector
Connections:	compressed air: Cu-pipe 8x1 Exhaust air line: Cu-pipe 15x1
Output:	at least 350 l/min at compressed air pressure of 700 to 1000 kPa
Use:	to drive compressed air driven implements in enclosed areas; spent air is passed into the open air via an exhaust air line.
Construction:	Take-offs for compressed air driven implements are connected to a separate piping network for 10 bar compressed air. Connection of the take-off to the implement is achieved via a coaxial hose. The inner hose passes compressed air to the implement, the

outer hose passes return air to the exhaust air duct.

Installation: for flush-mounting or hollow-wall installation (see below for installation dimensions), for installation in ceiling supply units or rail systems.

Product standard: DIN 13260-2

Warnings

- Prior to the use of the device, the user must be familiar with the product and the instructions for use and has to have understood them
- The gas outlet must be checked for external damage, contamination and the right gas type lettering before each use.
- When cleaning, make sure that no liquids enter the gas outlet. After cleaning, the gas outlet must be dried completely.
- Always keep all connections free of oil and grease! Risk of fire!
- After installation or maintenance of the gas outlet, an acceptance check according to DIN EN ISO 7396-1 must be performed.
- Only use the gas outlet with plugs of the respective gas standard.
- All serious incidents must be reported to Greggersen Gasetechnik GmbH and to the health authority responsible for the place of residence.
- Our products are partly made of brass with a lead content of more than 0.1%. As an alloy component, lead is firmly bound in the material so that no exposure is to be expected.

Function and operation

Only connectors for the Air Motor can be coupled into the take-off.

Push the connector into the take-off until an audible click is heard.

Coupling in the connector opens a valve. Compressed air flows through the supply line to the implement. The compressed air is used here and spent air fed back to the plug-in coupling. A line takes it from there to the exhaust air duct.

When the grip sleeve on the connector is withdrawn, the coupling comes undone and the connector can be taken out of the plug-in coupling. The valve closes the connection lines automatically and the compressed air supply stops.

Maintenance/Service

Adequate disinfection is obtained for all chrome-plated brass parts by wiping the outside of these parts. Plastic parts must be cleaned with a disinfectant solution. Dry take-offs and connectors completely before using them again.

Take-offs and connectors must be serviced at least once a year. The inlet and back pressure as well as the function of the extraction point must be checked using the air motor test device from the test kit. Faults and damage may be repaired only by authorised specialist staff. Only original GREGGERSEN spare parts are to be used for repairs.

900443 repair set Airmotor Forano

902114 Test kit for terminal units according 7396-1/2 DIN

902128 Test kit for terminal units according 7396-1/2 BS

902129 Test kit for terminal units according 7396-1/2 AGA

Assembly

The supply and exhaust air connections are connected according to the drawing on page 11.

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GREGGERSEN Gasetechnik GmbH / Bodestr. 27-31 / 21031 Hamburg / Germany