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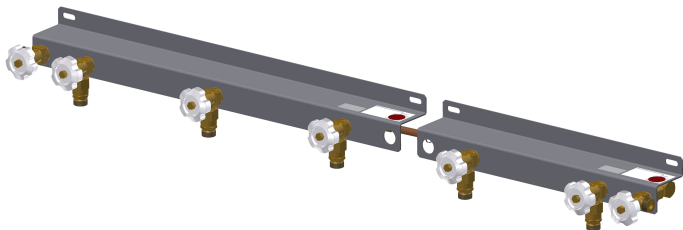
INSTRUCTIONS FOR USE

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High-pressure header

Deutsch	2
English	7

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High-pressure header

Dear customers,

Thank you for purchasing our Greggersen product.
If you have any questions or require information, please contact our sales and support team.

+49-(0)40 739357-0, sales@greggersen.de

Please read the instructions for use thoroughly before using the medical device. Only use it once you have completely understood the application and function!
All serious incidents must be reported to Greggersen Gasetechnik GmbH and to the health authority responsible for the place of residence.

Symbols:

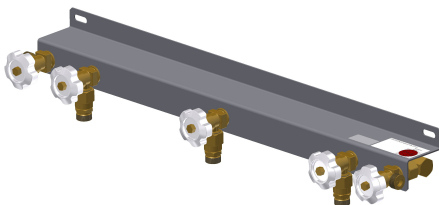
Date of manufacture
calender week year

Manufacturer

Keep free from
oil and grease

Intended use

For connecting one or more individual gas bottles to a bottle manifold.

Functionality

Each individual connection is equipped with a non-return valve so that backflow or emptying is prevented. A main shut-off valve with a G3/4" male connection enables the entire side to be separated from the supply and the collection line can be vented via a vent valve. All components are high-pressure resistant (BAM tested) and thus withstand the operating pressure of 20,000 kPa (200 bar).

Each individual or, in a double version, two bottles each are connected to the collecting line.

Variants

The manifold is available in the following versions:

- Number of input connections 1- to 10-fold
- Gas type-specific input according to DIN EN 477-1
- Connection elbows for making a connection between manifold and gas cylinder in single or double version with manual or wrench connection available separately
- 1 to 5 bottle holders available separately
- other versions and standards on request

Warnings

- Installation may only be carried out by authorized specialist personnel
- Only copper pipes according to DIN 1786 may be used for assembly
- the acceptance and handover of the gas supply system must be recorded
- Always keep all lines and connections free of oil and grease! Danger of explosion !!!
- When connecting the connection bends to the non-return valves, always counter the non-return valves with a wrench!

Installation

- Attaching the bottle holder to the wall
- Fastening the connection block to the wall
Mounting height usually 1700 mm from the middle of the hand wheel valves to the ground
- Lead the ventilation lines of the collecting lines to the outside
- Connection of the HP connection bends to the collecting line
Connection of the collecting lines to the mains or the switching system
- Flush the entire system after installation
- New check valves must be tightened with approx. 40 Nm
- !!! Danger !!! during soldering, the lines must be purged with inert gas

Separating gas cylinders

Before disconnecting a cylinder from the manifold elbows, all valves must be closed and the manifold vented. The sealing elements will be damaged if the connection between the gas cylinder and the connection bend is loosened with residual pressure in the system!

Connecting gas cylinders

Never use tools when connecting gas cylinders with manual connectors. This will damage the seals! It is sufficient to tighten the connection hand-tight. It must be ensured that the gas bottles are adequately secured with bottle holders.

After connecting the bottles, **slowly** open the valves!

Pressure surges must be avoided!

Danger of explosion!

Maintenance

Carry out a visual inspection of the collection line before each cylinder change (external damage). A function and leak test must be carried out at least once a year (if necessary: replacement of wear parts), and maintenance must be carried out every five years. The non-return valves must be replaced as part of maintenance. Malfunctions and damage may only be remedied by authorized specialist personnel. Only use original GREGGERSEN spare parts for repairs!

Disposal

There are no special disposal requirements for this product. The device can be disposed of in household waste. If the device is contaminated, appropriate precautionary measures must be taken.

There is no warranty claim for damage or malfunctions caused by improper transport, improper storage or improper use.

Accessories and Spare parts

Number	Description
900458	service set distribution bend DIN 477-1 oxygen
900503	service set distribution bend DIN 477-1 compressed air, carbon dioxide, nitrogen
900504	service set distribution bend DIN 477-1 nitrous oxide

Cylinder support

Number	Description
325300	Cylinder support for 1 cylinder
325304	Cylinder support for 2 cylinder
325305	Cylinder support for 3 cylinder
325306	Cylinder support for 4 cylinder
325307	Cylinder support for 5 cylinder
325869	Additional chain for cylinder support complete with 2 hooks

Non return valves

Number	Description
901351	High-pressure non return valve O ₂
901355	High-pressure non return valve AIR
901352	High-pressure non return valve N ₂ O
901353	High-pressure non return valve CO ₂
901354	High-pressure non return valve N ₂

Distribution bend with valve

Number	Description
324415	Distribution bend, single with vent valve

High pressure pipe

Number	Description
327304	High-pressure connecting-pipe

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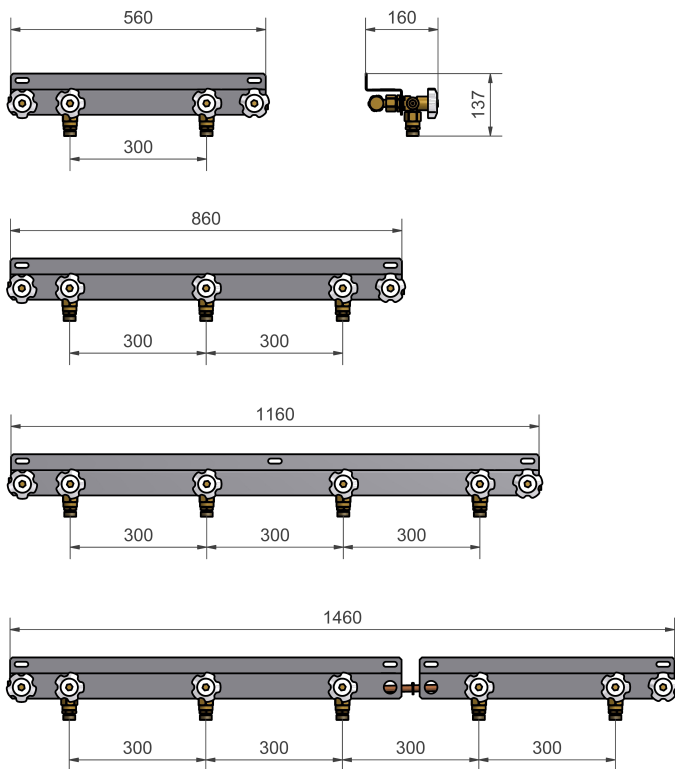
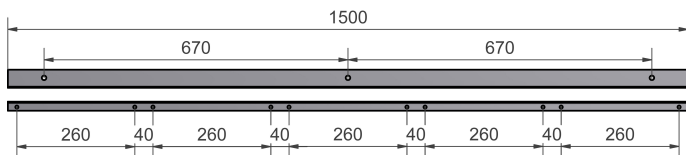
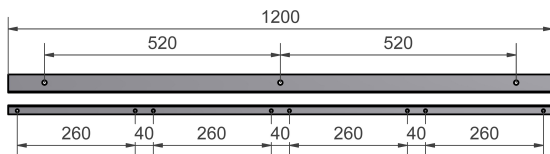
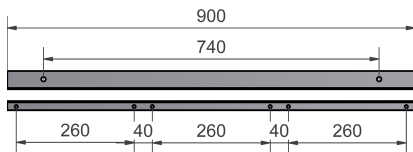
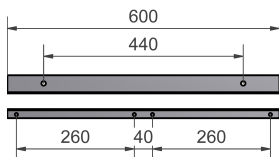
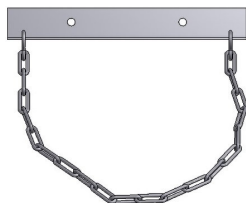
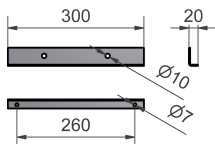
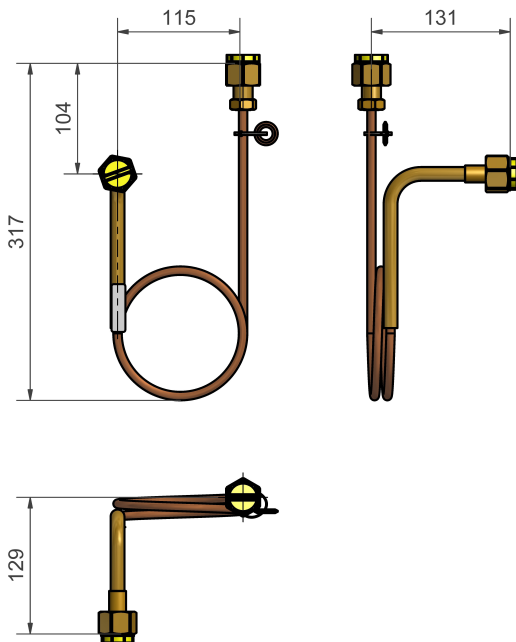
The dimensions of the larger high-pressure headers continue analogously.

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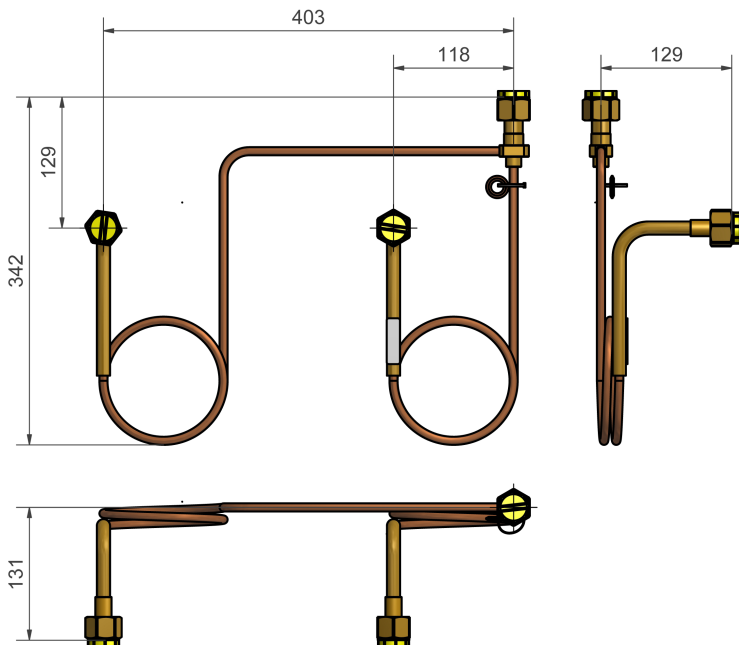
Depending on the connection standard and the type of gas, the dimensions vary slightly.

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GREGGERSEN Gasetechnik GmbH / Bodestr. 27-31 / 21031 Hamburg / Germany