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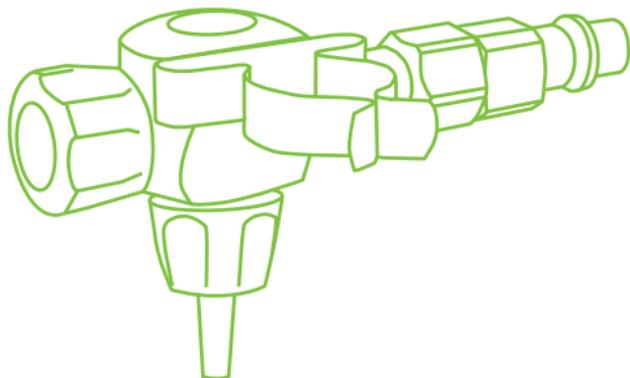
INSTRUCTIONS FOR USE

Verneblersteckventil

mit Klammer

Medical atomizer valve

with clamp

CE⁰⁴⁸²

Deutsch 2

English 6

Verneblersteckventil

mit Klammer

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with clamp

Dear customers,

Thank you for purchasing this Greggersen product.

Our sales and support team would be pleased to assist if you have any questions or need information.

Tel.: +49-(0)40 739357-0
e-mail: sales@greggersen.de

Please read the instructions for use thoroughly before using the medical device. Only use it once you have completely understood the application and function!

Medical device according to: Connection according to:

DIN EN ISO 15002
93/42/EWG

DIN 13260-2
BS 5682
NF S90-116
SS 875 24 30 (AGA)

Preface

The Medical atomizer valves from Greggersen Gasetechnik GmbH comply with the regulations of the Medical Device Directive 93/42/EEC and are classified as active, non-invasive devices of class IIa.

Variants

No.	Description
500370	Medical atomizer valve with clamp, DIN, AIR
900865	Medical atomizer valve with clamp, DIN, AIR/O ₂
901701	Medical atomizer valve with clamp, AGA, AIR
901812	Medical atomizer valve with clamp, BS, AIR
901902	Medical atomizer valve with clamp, NF, AIR

Intended use

For delivery of measured doses of medical compressed air or oxygen to a medication atomizer.

Functionality

The input plug is coupled into a gas outlet and thus supplied with a pressure of 400 to 500 kPa.

The gas flow to the outlet connection is opened with the hand wheel. The valve ensures a volume flow of 5 l/min. Various atomizer systems can be attached to the clamp when not in use.


Warnings

- Prior to the use of the device, the user must be familiar with the product and the instructions for use and has to have understood them.
- Before each use, the Atomizer Valve must be checked for external damage or contamination!
- Always keep all connections free of oil and grease!
Risk of fire!
- Do not use any tools when connecting or operating!
- All serious incidents must be reported to Greggersen Gasetechnik GmbH and the health authority responsible for the place of residence!

Device description

No.	Description
1	Plug
2	Clamp
3	Hand wheel
4	Hose connection

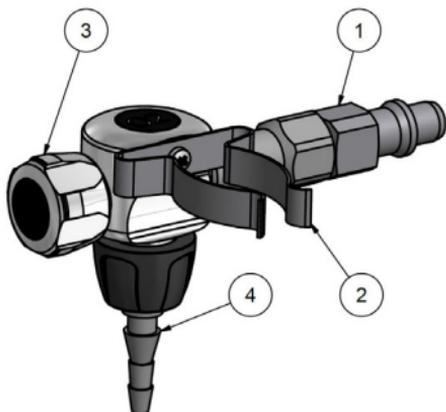Patient target group

The medical atomizer valve is used to supply gas with a defined volume flow to a medication atomizer. The use of the medication atomizer is determined by medical staff.

Intended user

The medical device is intended for clinical use by medically trained specialists (for example therapists, nursing staff, doctors).

Operation

An external visual inspection and a function test must be carried out before each use. During the external visual inspection, the atomizer valve must be examined for obvious damage or contamination. In the functional test, the device must be coupled to a gas outlet and the hand wheel opened. Check whether a gas is flowing out of the valve.

Before coupling the valve to a gas outlet, the hand wheel must be closed.

Couple the valve in the gas outlet and connect the medication atomizer. Open the hand wheel for operation.

The hand wheel must be fully open during operation.

If the hand wheel is only slightly open, the flow rate of 5l/min may not be achieved.

Close the hand wheel to end operation. The atomizer valve can remain in the gas outlet or can be removed from it. When not in use, the atomizer can be attached to the clip

Cleaning

The atomizer valves can be cleaned with a damp cloth using commercially available disinfectants and cleaning agents. When cleaning, make sure that no liquid gets into the device. Completely dry the device before it is used again. Health facility hygiene guidelines must be observed.

Maintenance

The atomizer valve must be serviced at least every 5 years. In the case of unusually heavy use, shorter maintenance intervals may be necessary. Maintenance and repairs may only be carried out by authorized expert personnel. Only use original **Greggersen** Gasetechnik GmbH spare parts.

Technical data

Primary pressure:	400...500 kPa
Types of gas:	oxygen (O ₂), compressed air (AIR / MA)
Standards:	DIN, BS, SS (AGA), NF
Output:	5l/min ± 20%
Temperature range:	-20°C to +60°C
Dimensions:	HxWxD: 85x75x120 mm
Weight:	250g

Accessories

No.	Description
900619	Hose connection 9/16" barbed connector for compressed gas

Notizen:

www.greggersen.com

GREGGERSEN Gasetechnik GmbH / Bodestr. 27-31 / 21031 Hamburg / Germany

17.05.2023/M.Oels/104015-02 DOKU - Vernebstecvk. m Klammer Greg-D/GB