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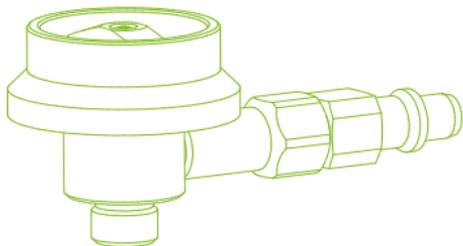
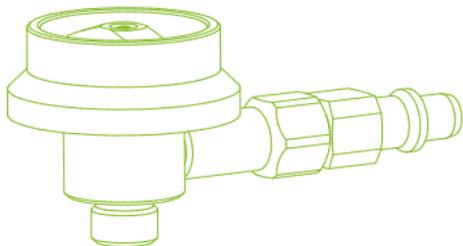
Nebuliser plug-in valve

Use

In association with the Kendall medicines nebuliser for metered output of medical gases for the inhalation of aerosol medicines.

Operation

Close the quick-acting gate valve and plug the plug-in valve directly into the take-off with the outlet downwards. Connect the Inhalette to the outlet and fill with the liquid aerosol solution.

For inhalation, pick up the Inhalette, open the quick-acting gate valve completely and begin inhalation. After use, close the quick-acting gate valve and place the Inhalette back on the retainer of the plug-in valve. The plug-in valve can be left in the take-off, ideally in the idle position. If the Inhalette is not being used, it can be plugged into the retainer of the plug-in valve.

Warnings

- Always keep all connections free from oil and grease!
- Never use tools when connecting the unit and the accessories; all connections can be tightened or loosened by hand.
- Do not use force to close the quick-acting gate valve: the valve seat may be damaged.
- The metering screw is coordinated with the admission pressure, gas type and the flow required. Other metering screws will affect the accuracy of the metering.
- The Nebulizer may only be used for the purpose outlined in this instruction of use and by briefed staff.

- Subject to technical change without notice; other designs upon request.
- Fluctuations with pre or counter-pressure, as well as the environmental conditions may have influence on the flow rate.
- There is no warranty for correct function in case of improper transport or storage or during operating the Kolibri not in accordance to this instruction of use.

Technical data

Type:	chrome-plated, all-metal construction, connector unit for compressed air, quick-acting gate valve
output:	5 l/min \pm 20% at 450 kPa \pm 50 kPa admission pressure
Connection Inlet:	plug-in connector units according to DIN 13260 part 2
Outlet:	9/16-18 UNF
Guideline:	DIN EN ISO 15002 DIN 13260-2 Medical Product Directive 93/42/ECC; Classification: IIa

Maintenance/Service

Adequate disinfection is obtained for all chrome-plated brass parts by wiping the outside of these parts. Plastic parts must be cleaned with a disinfectant solution. Dry the unit completely before using it again.

Prior to each use, make a visual check (external damages). The units should have a maintenance check at least once every five years (function and leak test), if the nebuliser is used to an unusually high degree, the intervals between maintenance might be shorter. Failures and damages may only be remedied by authorized expert personnel. Use only original **GREGGERSEN** spare parts!

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GREGGERSEN Gasetechnik GmbH / Bodestr. 27-31 / 21031 Hamburg / Germany