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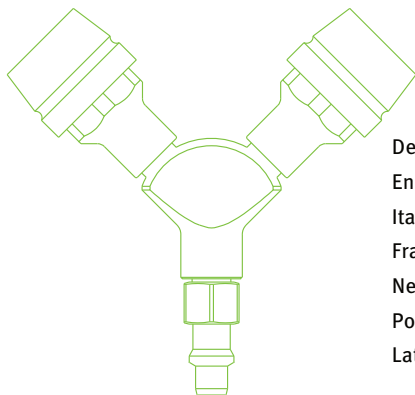
INSTRUCTIONS FOR USE

Y-Verteiler

Double-Extender

CE 0482

Deutsch	2
English	5
Italiano	8
Français	11
Nederlands	14
Polski	17
Latvian	20

Y-Verteiler

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Double-Extender

Dear customers,

Thank you for purchasing our Greggersen product.
If you have any questions or require information,
please contact our sales and support team.

+49-(0)40 739357-0, sales@greggersen.de

Please read the instructions for use thoroughly
before using the medical device. Only use it once
you have completely understood the application and
function!

All serious incidents must be reported to
Greggersen Gasetechnik GmbH and to the health
authority responsible for the place of residence.

Medical device in accordance with:

93/42/EWG
EN ISO 9170-1

NF S 90-116 BS 5682
SS 8752430 Medap

DIN EN ISO 13260-2

Symbols:

Consult the
instructions for use

manufacturer

Date of manufacture
calendar week year

Keep free from
oil and grease

Intended use

For connecting a maximum of two medical devices to a medical gas outlet.

Functionality

The Double-Extender is coupled to an outlet for compressed medical gases
or vacuum. This compressed gas or vacuum is provided via two extraction
points.

Variants

The Double-Extender is available in the following versions:

- Gas types O₂, AIR, AIR-800, VAC, CO₂, N₂O
- Gas standards DIN, BS, NF, SS (AGA), Medap
- in anti-magnetic design
- other versions on request

Warnings

- The total flow of the two gas outlets is limited by the output of the gas outlet into which the device is coupled.
- Never connect Y distributors in series
- Always keep all connections free of oil and grease! Fire hazard!
- No smoking or fire near oxygen equipment

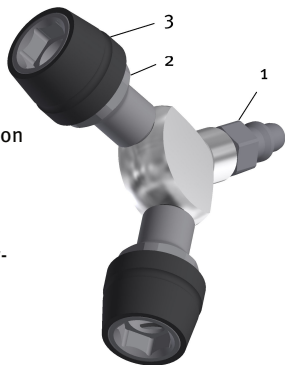
Drawing and parts list

1. Input connector
2. Gas outlet
3. Pressure ring

Service

Before each use, an external visual inspection and a functional test must be carried out. Couple the Y-distributor into the tapping point in such a way that the device is horizontal or that the plug-in couplings are next to each other. Up to two devices or low-pressure hose lines can then be coupled. After use, uncouple the devices or low-pressure hose from the outlets by pressing the pressure ring. Uncouple the Y distributor from the gas extraction point.

Cleaning

Wipe low-pressure hose lines at least once a month with a surface disinfectant based on the active substances aldehydes, quaternary ammonium compounds or alcohol. After cleaning, the product must be completely dried. The hygiene plans of the health facility must be observed.

Maintenance

The function and leak tightness of the Y-distributor must be checked at least once a year.

Disposal

There are no special disposal requirements for this device. The device can be disposed of in household waste. If the device is contaminated, use the appropriate precautions.

Technical specifications

permissible test pressure:	1400 kPa
Operating pressure:	400...500 kPa
Weight:	about 450g
Dimensions:	(WxHxD) 160 x 320 x 150 mm
Temperature range:	-10°C - 40°C
Lifetime:	10 years
Classification:	non-active, non-invasive, class IIa

There is no entitlement to rifle benefits in the event of damage or malfunctions caused by improper transport, improper storage or improper use.

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GREGGERSEN Gasetechnik GmbH / Bodestr. 27-31 / 21031 Hamburg / Germany