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
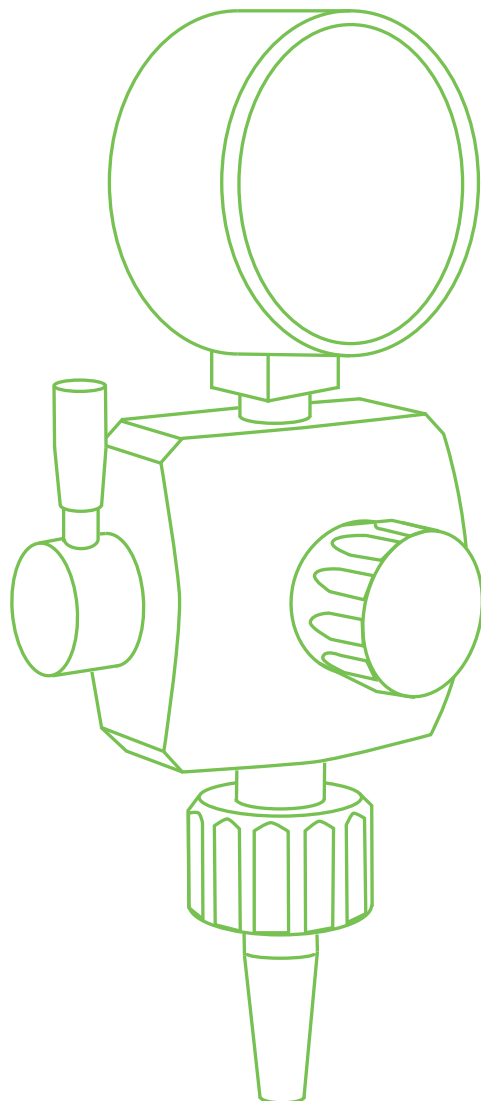
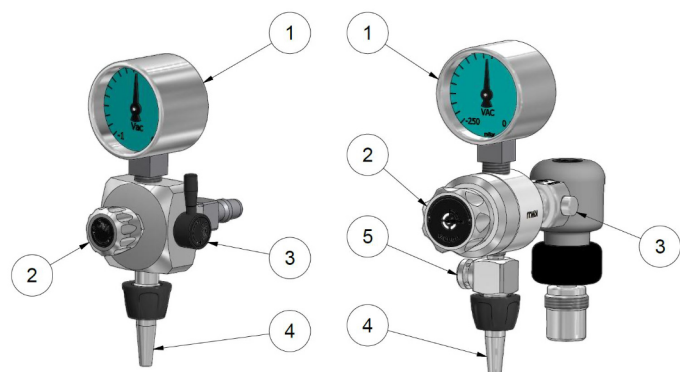
# Spatz Vacuum Regulator

## Dear customers,

we would like to thank you for purchasing this Greggersen Gasetechnik GmbH product. For questions and information, our sales and support team will be happy to assist you.

+49-(0)40 739 357-0; [sales@greggersen.de](mailto:sales@greggersen.de)

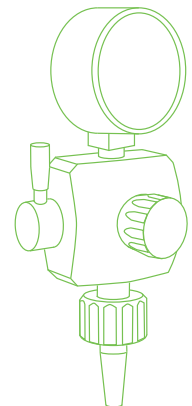Please read the instructions for use carefully before using the medical device. Use it only if you have fully understood the application and function. All serious incidents must be reported to Greggersen Gasetechnik GmbH as well as to the health authority responsible for the place of residence.

### Medical device according to:

DIN EN ISO 10079-3  
93/42/EWG

### Connections according to:

DIN 13260-2      SS 875 24 30 (AGA)    NF S90-116  
SN ENV 737-6:2005    DIN EN ISO 18082

## Symbols

	<p>Observe instructions for use</p>		<p>Do not use oil</p>
	<p>Date of manufacture, calendar week – year</p>		<p>Product contains lead</p>
	<p>Manufacturer</p>		<p>Order number</p>

## Use

To adjust the vacuum power for absorb medical liquids. The following accessories can be connected to the vacuum regulator:

## Accessories

- overflow bottle
- tube-connection 9/16

## Operation

Plug-in units can be directly plugged into an outlet; rail units are fastened on the medical instruments rail; vacuum supply is via the hose connection.

Prior to plugging the unit into the outlet, close the needle valve and the quick-action valve of the ejector regulator. For plug-in units, plug the unit into the outlet; and for rail units, plug the angle plug into the outlet. The vacuum regulator does not have any overflow safeguarding! Mandatory necessary for the mode: Operating equipment always use with a overflow bottle. With secret contaminated equipment is not repairable. Open the quick-action valve completely and adjust the desired vacuum with the needle valve. Suction can be interrupted any time with the quick-action valve without changing the set value.

Turn the valve to the left to adjust the desired vacuum. The safety valve will open when the adjusted vacuum is more than  $-16$  kPa “Sparrow -16” or  $-10$  kPa “Sparrow -10”; a pulsating sound will appear. After use, close the needle valve and the quick-action valve again. The vacuum regulator can remain in the outlet, preferably in park position.

## Warnings

- The vacuum regulator should only be operated by authorized personnel.
- Spatz -90: the bypass bore must always be free since the fine vacuum adjustment would otherwise be impaired.
- Never use tools when connecting the unit and the accessories; all connections can be tightened or loosened by hand.
- The vacuum regulator Sparrow -16 and -10 are limited by a safety valve; keep the holes free at all times; modification of the safety valve may influence the safety of the device.
- Do not forcibly close needle valve and quick-action valve: the valve seat may be damaged.
- Always operate the vacuum regulator with overflow safety: liquids or solids in the unit impair its performance.
- Subject to technical change without notice; other designs upon request.
- Fluctuations with pre or counter-pressure, as well as the site conditions may have influence on the flow rate.
- The product owner must ensure that all personnel handling the product are provided with the operating instructions & performance data and are fully trained to carry out that operation.
- There is no warranty for correct function in case of improper transport or storage or during operating the vacuum regulator not in accordance to this instruction of use.

## Technical data

Type:	chromium plated all-metal construction, plug- or rail unit, vacuum adjustment by valve, vacuum gauge, Sparrow -16 and -10 with safety valve
Capacity:	nominal range pressure gauge: 0 ... -100 kPa (Sparrow -90) infinitely adjustable between -20 ... -85 kPa
Capacity:	nominal range pressure gauge: 0 ... -100 kPa (High-Sparrow -90) infinitely adjustable between 0 ... -100 kPa
Capacity:	nominal range pressure gauge: 0 ... -25 kPa (Sparrow -16) infinitely adjustable between 0 ... -16 kPa (-160 mbar) vacuum is limited by safety valve
Capacity:	nominal range pressure gauge: 0 ... -16 kPa (Sparrow -10) infinitely adjustable between 0 ... -10 kPa (-100 mbar) vacuum is limited by safety valve
Gas:	vacuum
Operating pressure:	-99 kPa ... -40 kPa
Outlet:	9/16-18 UNF with vacuum tube-connection
Storage:	-30 ... +50 °Celsius
Use:	0 ... +40 °Celsius
Medical device:	Class IIa

## Maintenance / Service

External disinfection with a disposable cloth is sufficient for all chrome-plated metal parts. Clean plastic parts with a common disinfectant solution and follow manufacturers instructions. Let equipment dry completely before using it again.

Prior to each use, make a visual check (external damage).

A function and leak test must be performed at least once a year. Maintenance must be done every five years (replacement of wear parts).

Failures and damages may only be remedied by authorized expert personnel.

Do repairs only on the pressureless unit. Use only original GREGGERSEN spare parts.

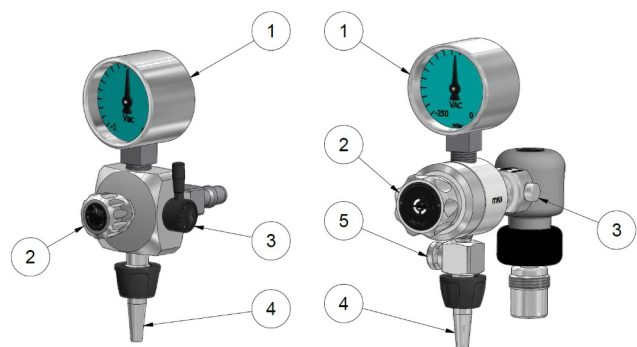
## Spare parts / accessories

### item no. description

900628	tube-connection 9/16" vacuum
900913	secretion overflow bottle, 0.25L

### pos description

1. vacuum gauge
2. needle valve
3. quick-action valve
4. vacuum nipple with nut
5. safety valve (only Sparrow -16 and -10)

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