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
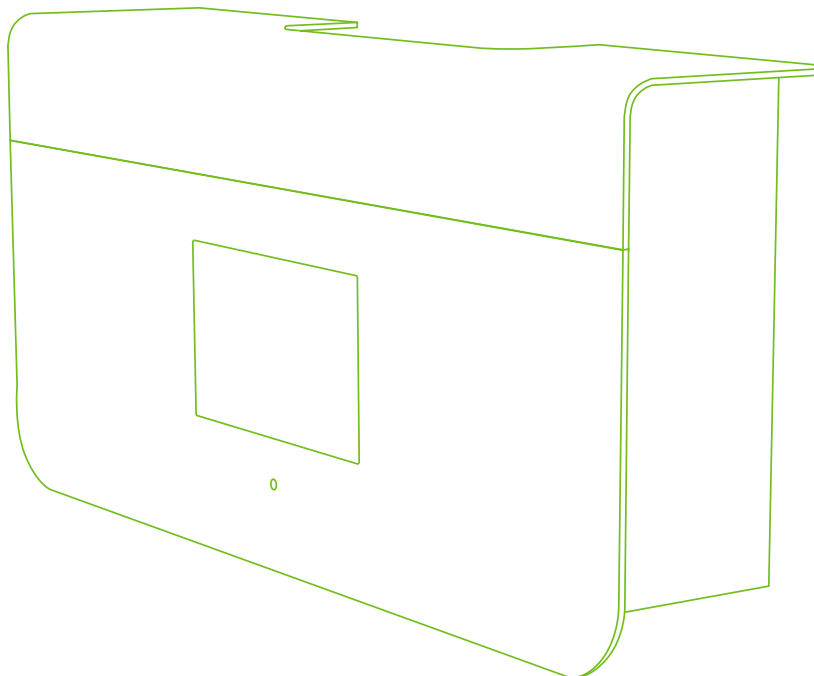
Zone monitor Source monitor

Dear customers,

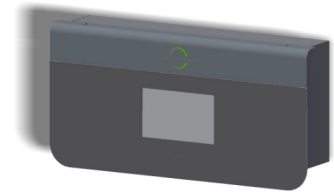Thank you for purchasing this Greggersen product.
If you have any questions or require information, please contact our sales and support team.

+49-(0)40 739 357-0, sales@greggersen.de

Please read the instructions for use thoroughly before using the medical device. Only use it once you have fully understood the application and function! All serious incidents must be reported to Greggersen asetechnik GmbH and to the health authority responsible for the place of residence.

Medical device in accordance with:

DIN EN ISO 7396-1

DIN EN 60601-1-2

DIN EN 62304

(EU) 2017-745

1. General information

The zone and source monitor from Greggersen Gasetechnik GmbH complies with the relevant provisions of the EU Medical Devices Regulation (MDR) 2017/745.

It is classified as a Class IIb active product.

Basis-UDI: 4031196MONITORW2

1.1 Intended use

Monitor for monitoring the status of supply systems for medical gases or vacuum in the professional healthcare sector in healthcare facilities with a central gas supply in accordance with DIN EN ISO 7396-1. The monitor is used to evaluate sensor values, shows them on a display and compares them with programmed limit values. In the event of misconduct, the staff is alerted optically and acoustically. Messages are documented in a logbook. All information are provided via a digital interface.

1.2 Combination with other products

The Ethernet interface enables seamless integration into the Greggersen CEGAnet system, ensuring central monitoring of all states. The optional Modbus or relay modules can also be used to forward data to external monitoring systems.

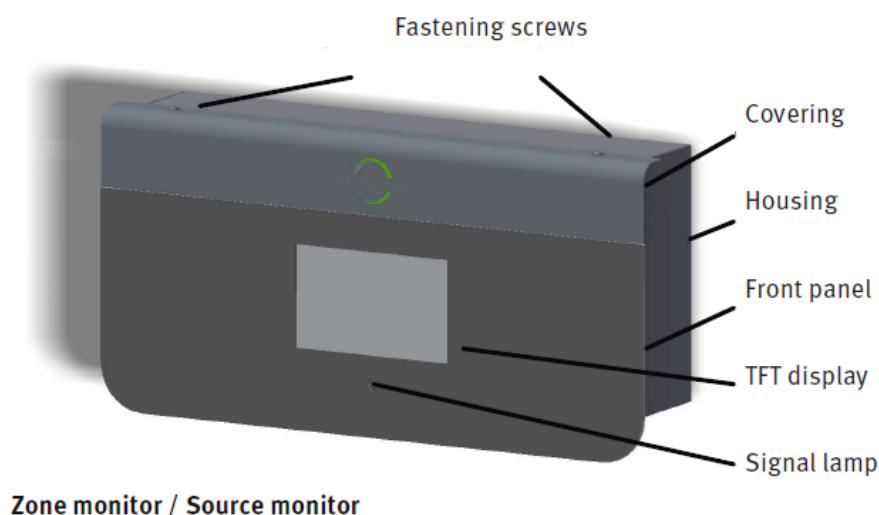
1.3 Warnings

- Prior to using the device, the user must be familiar with the product and the instructions for use and has to have understood them.
- Our products are partly made of brass with a lead content of more than 0.1%. As an alloy component, lead is firmly bound in the material so that no exposure is to be expected.

2. Device description

The zone and source monitors are configured according to the customers requirements for the respective area. All monitors consist of a housing made of powder-coated metal and can be surface-mounted. Zone monitors are used to monitor the line pressure of medical gases or vacuum in a defined zone such as a nursing ward, an operating theatre or an intensive care unit. Source monitors monitor states and signals from a medical gas supply center. Up to six analog sensors (4-20 mA) as well as eight digital signals (potential-free contactor) can be connected to the monitor electronics. The sensor data are processed in the electronics and compared with the individually specified limit values of the health facility. The measured data or states are shown on the TFT display. In the event of a fault, the staff will be alerted visually and acoustically. To forward the various signals and warnings to a building control system, the basic zone monitor board can be expanded with up to five relay modules. In the event of a power failure, this will be indicated by a signal lamp on the front panel. The device is equipped with a battery for this purpose. The menu language is German, French, Dutch, Italian, Danish, Latvian or Hungarian, depending on the customer's request. To access the inside of the monitors for maintenance or installation, remove the top cover by removing the screws.

3. Installation and commissioning

The installation and commissioning must be carried out by qualified personnel. Before each initial and re-commissioning, the system must be subjected to a commissioning test in accordance with DIN EN ISO 7396-1. The function of the individual alarms, the correct configuration of the alarm thresholds and, if necessary, the forwarding of alarm signals must be checked. In addition, the test must be carried out in accordance with the applicable country-specific regulations for electrical devices.

3.1 Checking delivery

Prior to installation, the product must be checked for damage to the transport packaging, for obvious damage to the product, and for the completeness and correctness of the configuration according to the delivery note.

3.2 Required tools and aids

To install the monitor, only ordinary tools, such as drills, cordless screwdrivers, Allen keys, spirit levels, wire strippers and multimeters, are required. A spatula and filler compound may be required to close the gaps.

3.3 Installation

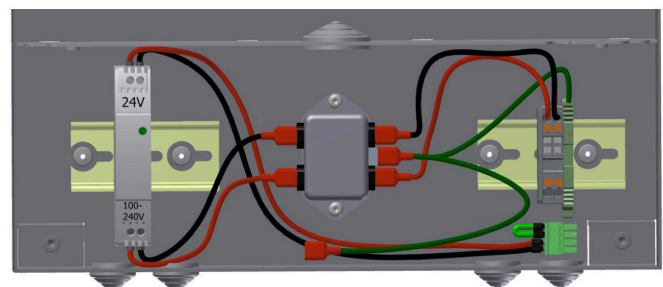
Remove the housing from the packaging and disassemble the electronics. To do this, remove the monitor cover by removing the fastening screws. Then loosen the screws above the display, tilt the electronics forward and remove by lifting slightly. Cut free the bushings for electrical cables in the rear section.

Install the housing horizontally on the wall using the holes in the housing. Insert the electrical cables into the housing. It must be ensured that the wall is sufficiently load-bearing.

3.4 Electrical connection

The electrical connection or dismantling may only be carried out in a de-energised state! Strip the cable ends 8-9 mm. Use wire-end ferrules for flexible cables.

In order to ensure continued operation in the event of a power failure, the device must be connected to the emergency power supply. It must be ensured that the supply line is professionally secured.

Suggestion for cable selection

Mains voltage terminal blocks:

NYM- 3x1,5 mm²- to 3x2,5 mm²

Analog/Digital inputs:

LiYCY- 2x2x 0,5 mm² (flexible)

Do not connect shield

IStY- 2x2x0,6 mm² (rigid)

Relaismodule:

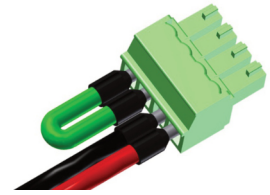
IStY- 2x2x0,6 mm²,

IStY- 6x2x0,6 mm²

Network cable:

at least CAT 6


Note: By plugging in the power plug and closing the jumper, the optical alarm is enabled with an LED in the event of mains voltage failure. To prevent the internal battery from discharging too quickly, disconnect the power plug or remove the jumper during transport, while starting operation or other prolonged periods without power supply.

3.4.1 Installation of the electrical components

1. Install the terminal blocks on the DIN rail
2. Fasten line filter with screws
3. Clamp power supply unit to the DIN rail
4. Install connecting cables

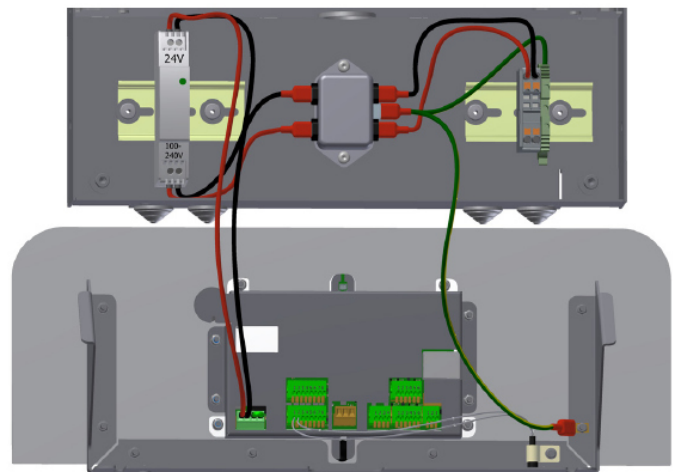
Caution! When connecting the connecting cable between the power pack and the electronics, make sure that the polarity is correct. If mixed up, the electronics can be damaged.

3.4.2 Connection of sensors and relay module

1. Connect the sensor cables to the electronics.
The terminal assignment is shown on the back of the electronics
2. Fasten the line filter with screws
3. Clamp the power supply unit to the DIN rail
4. Install the connecting cables

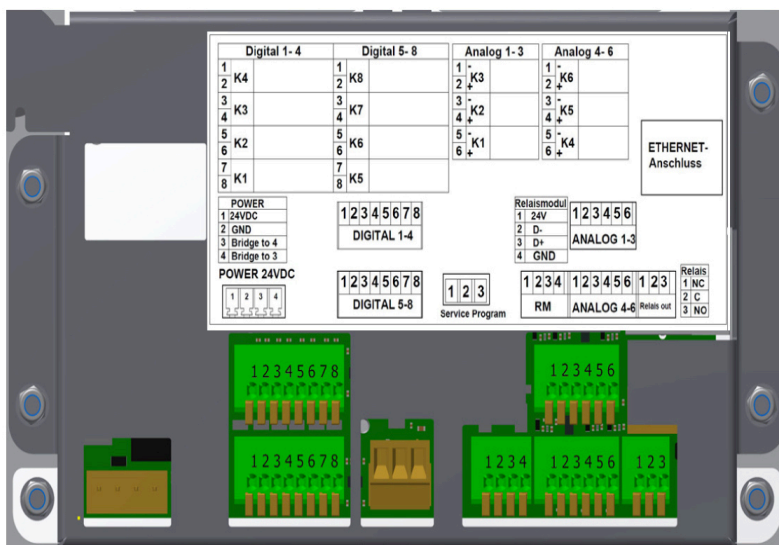3.4.3 Pin Assignment

| Pin | Function |
|-----|----------------|
| 1 | + 24 VDC |
| 2 | GND |
| 3 | Bridge Battery |
| 4 | |

Supply voltage

| Pin | Function |
|-----|----------|
| 1 | NC |
| 2 | C |
| 3 | NO |


Main board relay

| Pin | Function | Pin | Function | Pin | Function | Pin | Function | Pin | Function | Pin | Function |
|-----|-----------|-----|-----------|-----|------------|-----|------------|-----|----------|-----|----------|
| 1 | Channel 4 | 1 | Channel 8 | 1 | Channel 3- | 1 | Channel 6- | 1 | Relay 1 | 1 | Relay 6 |
| 2 | 4 | 2 | 8 | 2 | Channel 3+ | 2 | Channel 6+ | 2 | | 2 | |
| 3 | Channel 3 | 3 | Channel 7 | 3 | Channel 2- | 3 | Channel 5- | 3 | Relais 2 | 3 | Relay 5 |
| 4 | 3 | 4 | 7 | 4 | Channel 2+ | 4 | Channel 5+ | 4 | | 4 | |
| 5 | Channel 2 | 5 | Channel 6 | 5 | Channel 1- | 5 | Channel 4- | 5 | Relay 1 | 5 | Relay 4 |
| 6 | 2 | 6 | 6 | 6 | Channel 1+ | 6 | Channel 4+ | 6 | | 6 | |
| 7 | Channel 1 | 7 | Channel 5 | | | | | | | | |
| 8 | 1 | 8 | 5 | | | | | | | | |

Channel 1-3 Channel 4-6 Channel 1-3 Channel 4-6 Channel 1-3 Channel 4-6

| Pin | Function | Colour cable |
|-----|----------|--------------|
| 1 | + 24 VDC | red |
| 2 | D - | whit |
| 3 | D + | green |
| 4 | GND | black |


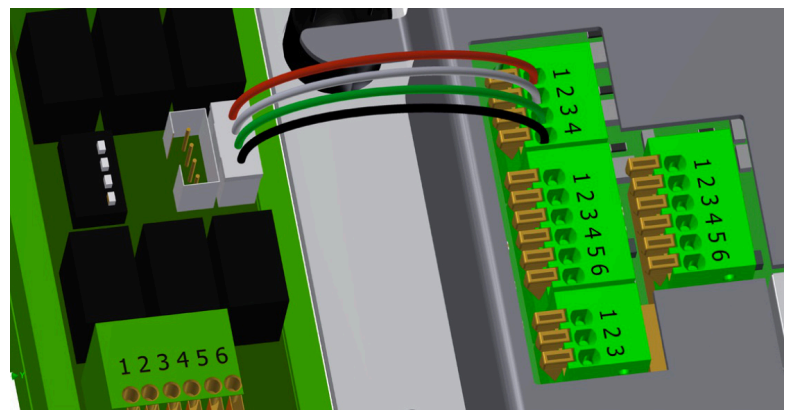
Relaismodul

Relay module connection

| RM | S ₁ | S ₂ | S ₃ | S ₄ |
|----|----------------|----------------|----------------|----------------|
| 1 | OFF | OFF | OFF | OFF |
| 2 | ON | OFF | OFF | OFF |
| 3 | OFF | ON | OFF | OFF |
| 4 | ON | ON | OFF | OFF |
| 5 | OFF | OFF | ON | OFF |

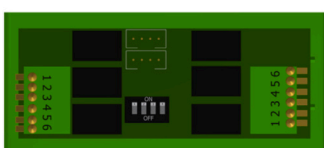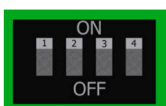Relay module connection


Dip switch

4. Operation


Before setting up, the displayed values of the individually monitored media must be checked for plausibility. By varying the line pressure of individual gases, for example, you can check whether the individual sensors are correctly assigned to the displayed messages. The zone and source monitor is designed to be operated intuitively. The zone monitor with touch screen has several menu levels. In the main view, each channel is displayed with the current pressure value or status. The keys in the menu line can be used to access the Logbook or to enter the Settings range.

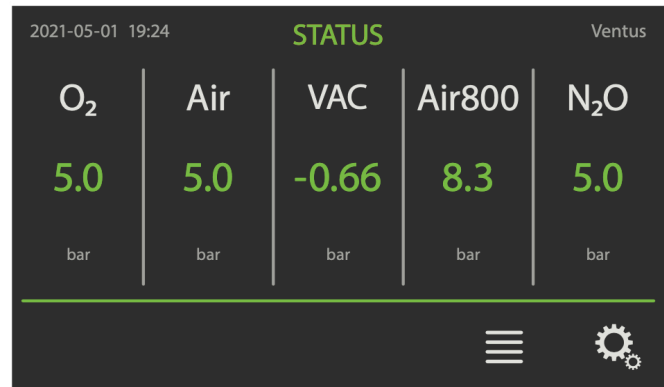Also see 8. Symbols

4.1 Alarm and warning signals

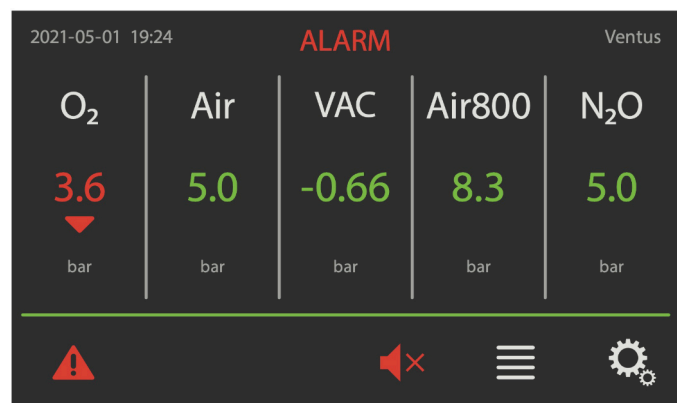Depending on the configuration, the monitor responds to events such as deviating pressure values or opening of the door, with visual and/or acoustic signals. A signal is displayed as long as the cause of the event continues. If the cause is eliminated, this signal also disappears.

Display „green“: Everything is OK. No actions are necessary.

Display „yellow“: An event has occurred that requires necessary action in the foreseeable future.

Display „red“: An event has occurred that requires immediate action. (e.g. line pressure outside the limit values). In the event of alarms, an acoustic alarm also sounds, which, depending on the setting, can be suppressed for up to 15 minutes by pressing the mute button.

Note: When monitoring vacuum, it may occur that the lower limit value (- 1.0 bar) is exceeded and a warning „Vacuum too high“ is displayed. This event is documented in the logbook, but there is no acoustic signal tone and this warning is not forwarded!

4.2 Logbook

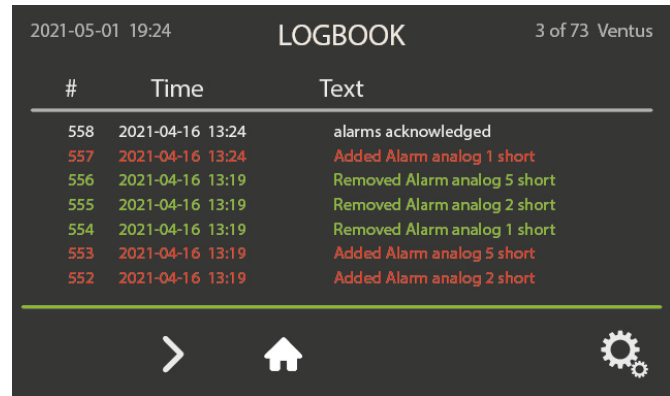
The logbook chronologically lists all events that occurred during operation. Events are alarms, warnings or also changes of device settings.

Use the arrow keys to navigate to older or earlier events.

Each event is assigned a consecutive number, date and time, as well as an alarm text.

Alarms are displayed in red, warnings in yellow, informations in green, and system signals in white.

For analog or digital input events, an assignment to the respective channels is made with the configuration document, which is part of the product documentation.


| # | Time | Text |
|-----|------------------|------------------------------|
| 558 | 2021-04-16 13:24 | alarms acknowledged |
| 557 | 2021-04-16 13:24 | Added Alarm analog 1 short |
| 556 | 2021-04-16 13:19 | Removed Alarm analog 5 short |
| 555 | 2021-04-16 13:19 | Removed Alarm analog 2 short |
| 554 | 2021-04-16 13:19 | Removed Alarm analog 1 short |
| 553 | 2021-04-16 13:19 | Added Alarm analog 5 short |
| 552 | 2021-04-16 13:19 | Added Alarm analog 2 short |


4.3 Settings

The set parameters of the monitor and the individual channels are displayed in the Settings menu. Changes are only possible with the CAScon cable USB-RS-485.

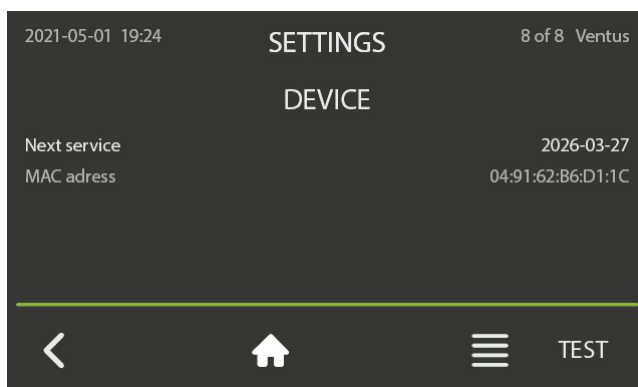Use the arrow keys to switch between the individual channel or device settings. Press the Home button to switch back to the main view.

| SETTINGS | | 8 of 8 Ventus |
|--------------|-------------------|---------------|
| DEVICE | | |
| Next service | 2026-03-27 | |
| MAC adress | 04:91:62:B6:D1:1C | |
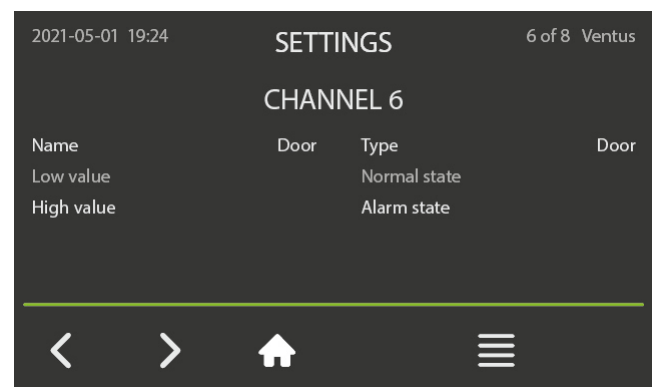
| SETTINGS | | 6 of 8 Ventus |
|------------|--------------|---------------|
| CHANNEL 6 | | |
| Name | Door | Type Door |
| Low value | Normal state | |
| High value | Alarm state | |

4.4 Set date and clock

The U-setting is located under SETTINGS. With arrow key (right arrow) all the way to last page. Click on the clock here. Press arrow up or down to set the clock. When the setting is complete press against clock symbol as confirmation. Back to main screen.

5. Cleaning

The outside of the zone or source monitor can be wiped with a slightly moistened soft cloth. Commercially available cleaning agents and disinfectants can be used. Care must be taken to ensure that no liquid gets inside the product. Do not use cleaning agents or disinfectants that are not suitable for common plastic or metal surfaces.

6. Maintenance and repair

Every zone or source monitor must be visually inspected at least once a year.

The device must be checked for external or obvious damage, legibility of the displays and plausibility of the displayed values. In addition, a test run must be carried out by pressing the Test button. The Test button is located on the last page under Settings. During the test run, all relays of the electronics are enabled, which can trigger alarms for central monitoring.

The battery must be replaced every 5 years or in the event of premature discharge. The information in the battery spare parts kit must be observed for this purpose!

Maintenance and repairs must only be carried out by authorised expert personnel. Only use genuine Greggersen spare parts when performing repairs.

All maintenance, tests or repairs carried out on the product must be recorded, including any anomalies identified and the associated test results.

7. Parameterization

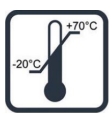
The electronics can be programmed using of GAScon software and the CAScon cable USB-RS-485. The separate parameterization instructions must be observed here.

8. Symbole

| Symbol | Function | Meaning |
|---|---------------|--|
|  | Previous page | Go to previous page |
|  | Next page | Go to next page |
|  | Settings | Calls up the system and channel settings |
|  | Test | Automatic system test |
|  | Home | Go to main view |
|  | Logbook | Call logbook |
|  | Mute | Interruption of the acoustic alarm |
|  | Caution | new alarm signals |
|  | cable break | Cable is broken |
|  | Short circuit | Cable is damaged |

9. Markings

| | | | |
|---|----------------|---|-------------------------|
|  | Order number |  | Protection class I |
|  | Medical Device |  | top |
|  | Manufacturer |  | Temperature conditions |
|  | Not waterproof |  | Follow the instructions |
|  | Not drop proof | | |
|  | Not cut-proof | | |

10. Technical data

| | |
|--------------------------------|---|
| Device protection class: | 1 |
| Protection type: | IP 40 |
| Software security class: | A |
| Inputs: | 6 analog 4-20mA for sensors 8 digital for contact switches |
| Outputs: | potential-free relay (collective signal), Ethernet interface Optional relay module (up to 30 potential-free contacts) |
| External supply voltage: | 110 VAC - 230 VAC (50 - 60 Hz) |
| Internal supply voltage: | 24 V DC |
| Power consumption electronics: | max. 0,5 A |
| Capacity relay module: | 30 V DC / 1 A bzw. 125 V AC / 0,6 A |
| Dimensions: | 189 x 390 x 85 mm (HxBxT) |
| Operating temperature: | +10°C - +40°C |
| Storage temperature: | -10°C - +70°C |
| Humidity: | 15 % - 95 % |
| Air pressure: | 700 hPa - 1.060 hPa |
| Expected product lifetime: | 10 years |

The accuracy of the measured values essentially depends on the accuracy of the sensors used. The pressure values are displayed with one digit behind the decimal point, or with two digits for vacuum.

11. Disposal

This product must not be disposed of with household waste in accordance with the Waste Electrical and Electronic Equipment Directive (2012/11/EU) and national laws. This product must be handed over to a designated collection point for the recycling of waste electrical and electronic equipment.

12. Spare parts

| Spec. No. | Designation |
|-----------|--|
| 905294 | CAScon cable USB-RS-485 |
| 905295 | Battery CAS CR2032 |
| 905292 | Relay module basic |
| 905293 | Relay module extension |
| 106042 | Mains filter Schaffner FN2030A-1-06 |
| 104922 | DIN rail power supply unit 24V DC / 0.5A |

13. Error messages

| Error message | Possible causes | Action |
|----------------|--|--------------------|
| Low battery | Battery is empty or too old | Change battery |
| No network | Ethernet line has no connection | Check connection |
| Low supply | The voltage source is less than 22 V (e.g.: power failure, power supply defective) | Check power supply |
| | Incorrect value set in the configuration | Notify Service |
| Internal error | Error in program code basic module | Notify Service |
| Bus error | Error in program code relay module | Notify Service |
| CR Alarm | CRC error in the EEPROM | Notify Service |
| Cable cut | Connection line interrupted | Check connection |
| | Sensor defective | Change sensor |
| | Incorrect value set | Notify Service |
| Short circuit | Short circuit connecting cable | Notify Service |
| | Sensor defective | Change sensor |
| | Incorrect value set | Notify Service |
| RM comm. error | Connecting cable interrupted | Check connection |

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