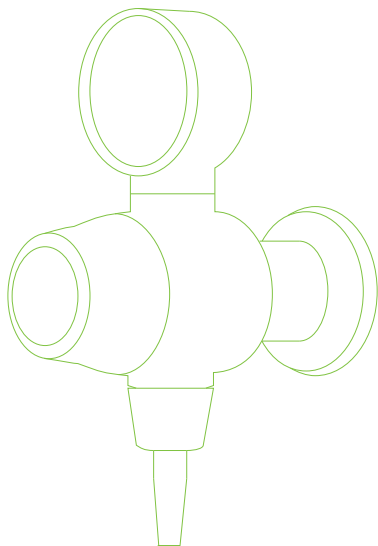
**GEBRAUCHSANWEISUNG
INSTRUCTIONS FOR USE
ISTRUZIONI PER L'USO
MODE D'EMPLOI
ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ**

PIROL

CE 0482

Deutsch	2
English	7
Italiano	12
Français	17
Русский	22

Vakuumregler Pirol

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Vacuum regulator Pirol

Dear customers,

Thank you for purchasing our Greggersen product. If you have any questions or require informations, please contact our sales and support team.
+49-(0)40 739357-0, sales@greggersen.de

Please read the instructions for use thoroughly before using the medical device. Only use it if you have completely understood the application and function! All serious incidents must be reported to Greggersen Gasetechnik GmbH and to the health authority responsible for the place of residence.

Medical device in accordance with:

93/42/EWG DIN EN ISO 10079-3 DIN EN ISO 13260-2
DIN EN ISO 19054 DIN EN ISO 18082 Carbamed

Symbols

Consult the instructions for use

manufacturer

Date of manufacture calendar week year

Keep free from oil and grease

Intended use

The Pirol vacuum regulator is used to provide vacuum to downstream medical devices. Plug-in devices are connected to a vacuum terminal unit. Rail devices are secured to a device rail and are supplied with vacuum via a low-pressure hose system. The Varioport and Conport device variants can also accept two suction containers. A hand wheel is used to regulate the vacuum to the required value. The vacuum is provided at an inlet connection or at a barbed inlet connector. The medical personnel decides on use in the Professional Healthcare area in medical institutions. The regulator must not be used for low-vacuum drainage (e.g. thoracic drainage), for syringes, for anaesthetic gas scavenging systems, for laboratory suction, for autotransfusion systems, for ventilators (in obstetrics) or for smoke extraction systems.

Functionality

The Pirol vacuum regulator works according to the principle of spring-loaded membrane regulation. After connecting the device to the central vacuum supply via the gas type and standard-specific interface (DIN, BS, NIST, etc.), the desired vacuum can be set by the user using the hand wheel. Turning the hand wheel preloads a spring coupled to an elastic diaphragm. The vacuum generated by the controller is formed by the balance of forces from the central vacuum and the spring force. The resulting air gap between the ram head and the central bore enables the air to be transported away into the central gas supply and thus for suction.

Variants

The Pirol vacuum regulator is available in the following versions:

- Power levels -90 and -30 kPa
- Plug devices according to DIN, Medap and Carbamed
- Rail devices for DIN rails and Swiss standards

Plug device

Rail device DIN

Rail device Switzerland

Warnings

- The vacuum regulator can only be used by trained medical personnel.
- Not suitable for outdoor operation and/or rescue operation.
- Not suitable for use for drainage in the low vacuum range (e.g. chest drainage).

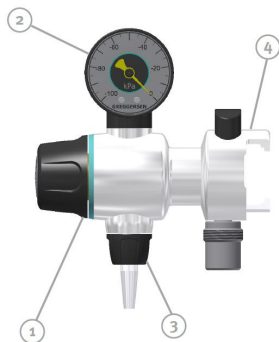
- For the operation is an secretion overflow bottle with backflow protection required.
- Fluctuations in the upstream pressure and the ambient conditions can affect the flow rate.
- Over-suctioned devices are irreparable and are not allowed to be used be used.

Drawing and part list

1. Hand wheel
2. Manometer
3. Vacuum nozzle
4. Plug or Rail clamp

Operation

- Before every use, a functional test and external visual inspection has to be carried out to ensure the proper function of the device. Close the vacuum flow, open the hand wheel valve and check whether the device builds up a vacuum. This can be read from the manometer.
- The device must be switched off to connect to the gas supply (fine metering valve closed).
- Do not operate the fine metering valve with force. The regulator unit can be damaged. The valve is closed as soon as a noticeable resistance builds up on closing the fine metering valve.
- Never use tools for connecting the accessories! All connections can be tightened or released by hand. Connecting and releasing the accessories may only be carried out in the depressurised state.
- To avoid malfunctions or faults in operation, the device must only be operated with the outlet directed downward!

Starting up

Plug-in unit:

Couple the vacuum regulator with the connector into the terminal unit. Observe the gas type marking on the label. Ensure that the device is fully engaged.

Rail-mounted unit:

Lock the vacuum regulator in place with a rail clamp on a rail. Ensure that the rail clamp is fully engaged. Then connect the device with the compressed air source via the NIST connection.

Operation

The required vacuum is set via the hand wheel valve and can be read from the manometer. The suction hose is connected at the vacuum nozzle. The manometer can be turned by 360° for improved visibility. The vacuum builds up at different rates depending on the volume of the downstream devices.

Taking out of service

After using the device, the fine metering valve has to be closed again.

As the vacuum is measured at the outlet, the manometer display does not always go to “zero”. Only if the components connected are relieved (opened) does the display go to “zero”.

The vacuum regulator can remain in the terminal unit or can be disconnected. Hold the device firmly while disconnecting from the terminal unit!

Cleaning

After each use it is recommended to clean the device with wipe disinfection. Use commercially available disinfectant for this purpose.

The device must be fully dried prior to subsequent use.

Do not soak the device with disinfectant! The regulator unit can be damaged.

Maintenance and service

- Maintenance has to be performed every 5 years.
- An inspection with a functional test and a leak test has to be performed at least every 2 years.
- In case of unusually heavy-duty use, shorter servicing and inspection intervals may be required.
- Failures and damages may only be remedied by authorized expert personnel.
- Repairs can only be carried out on the depressurised device.
- Use only original GREGGERSEN spare parts!
- There is no entitlement to any warranty claims in the event of damage or malfunction arising from improper transport, improper storage or other than the intended use.

Technical data

Storage:	-30°C ... +50°C	
Use:	0 ... +40°C	
Gas type:	Vacuum / VAC	
Prim. pressure:	-40 ... -99 kPa	
Inlet:	9/16"-18 UNF with nozzle for vacuum-devices	
Pirol -30:	Performance class: medium vacuum	
	adjustable: 0 ... -30 kPa ± 10%	
	Volume flow: > 20 l/min	
Pirol -90:	Performance class: high vacuum / high volume flow	
	adjustable: 0 ... -90 kPa ± 10%	
	Volume flow > 20 l/min	
Weight:	Plug device:	650 g
	Rail device DIN:	795 g
	Rail device Switzerland:	850 g
Class:	IIa	

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GREGGERSEN Gasetechnik GmbH / Bodestr. 27-31 / 21031 Hamburg / Germany